

Disposable Masks

(colour: blue / black / white)

SKU: 7224 | 7225 | 7226

INTENDED USE

This product is intended for infection control purposes in healthcare and standard professional/domestic industries. When worn properly, it helps minimise contamination caused by exhaled microorganisms and reduce the potential exposure of the wearer to blood and body fluids.

SIZING SPECIFICATION

The mask body is 17.5cm x 9.5cm, with a deviation of 0.5cm.

COMPONENTS AND STRUCTURE

This product consists of a 3 layer mask body, nose clips, and ear loops. The material in both the top and bottom layer of the main body is leak proof polypropylene spunbond non-woven fabric. The middle layer is a melt-blown inner core. The nose clamp is made of iron wire with a plastic coating, and the ear loops are made of urethane elastic fibre.

PRODUCT PERFORMANCE

This product is a Type IIR surgical mask and is in full conformity with EN 14683:2019. The mask is 3-ply, with different materials and thicknesses outlined above, to accurately reach Type IIR standard (EN 14683:2019). The main properties of the masks are as followed:

- Splash Resistance > 120mmHg
- Bacterial Filtration Efficiency (BFE) > 98%
- Latex free


WARNING

Make sure the product is complete and within the expiration date prior to wearing. Confirm both the nose and chin are inside the mask once the mask is on the face, to gain the expected and best protection. Individual masks should only be used once, and changed immediately if contaminated with blood or body fluid. Do not use if allergic to polypropylene spunbond non-woven fabric. Use this product as soon as possible after opening.

STORAGE AND TRANSPORTATION

Type IIR Surgical Face Masks have a Bacterial Filtration Efficiency of > 98% and are splash resistant to a pressure > 120mmHg. They are for single use only and should be worn for a maximum of four hours. The product is disposable, non sterile and valid for three years. Prior to use, please pay attention to the expiry date. The product should be stored in a cool, dry place with good ventilation, as well as in a clean environment with good ventilation.

MANUFACTURE DATE

Please find the date on the back of the box next to 

INSTRUCTIONS FOR WEARING

Make sure to wash your hands before touching the mask. Keep the coloured side on the outer side. With the nose clip at the top, place the mask over the mouth and nose. Hang the ear loops on each ear separately. Press the nose clips firmly in place until comfortable.



Clean your hands with soap and water or hand sanitizer before touching the mask



Hold the mask by the ear loops and place a loop around each ear



Pull the bottom of the mask over your mouth and chin



Mold or pinch the stiff edge to the shape of your nose






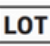


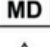




Avoid touching the front of the mask when wearing



Throw the mask in the trash

SERVICE LIFE 3 years.

EXPLANATION OF SYMBOLS AND LABELS

Symbols	Title of symbol	Explanation
	Manufacturer	Indicates the medical device manufacturer.
	Date of manufacture	Indicates the date when the medical device was manufactured.
	Authorised representative in the United Kingdom	Indicates the authorised representative in the United Kingdom
	Batch code	Indicates the manufacturer's batch code so the batch or lot can be identified.
	CE mark	Indicates the medical device was approved in the European Community.
	Use by date	Indicates the date after which the medical device is not to be used.
	Medical device	Indicates this item is a medical device.
	Non-Sterile	Indicates a medical device that has not been subjected to a sterilisation process.
	Do not reuse	Indicates a medical device is intended for one use, or for use on a single patient during a single procedure.
	Natural rubber latex is not present	Indicates natural rubber or dry natural rubber latex is not present as a material of construction within the medical device or the packaging of a medical device.
	United Kingdom Conformity Assessment	Indicates the Medical Device was registered and approved for sale in the Great Britain.

The Declaration of Conformity is available on the product page on www.dtsprotect.com

EC Rep: Medpath GmbH, Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

Product manufactured by: DTS Trading Inc. Ltd
 Unit 1, Chiltern Vale Farm, Coopers Court Rd, Stokenchurch, United Kingdom, HP14 3UE